





Application: Used in male circumcision for both adults and children



Special design of 60° sharp corner staples
Better penetration of the tissue, which brings better staple formation,
and better hemostasis effect.

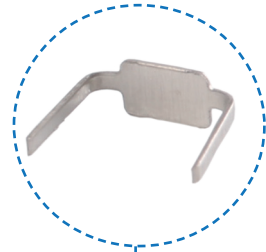


Stainless steel balanus cover,
make the formation more stable

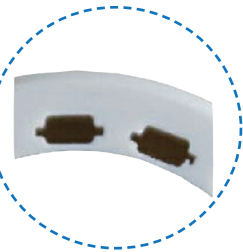


Design of anvil slot sealing edge,
conductive to better stapler formation,
and better desirable hemostatic effect




Segmented silicon pad,
reduce postoperative discomfort and facilitate healing

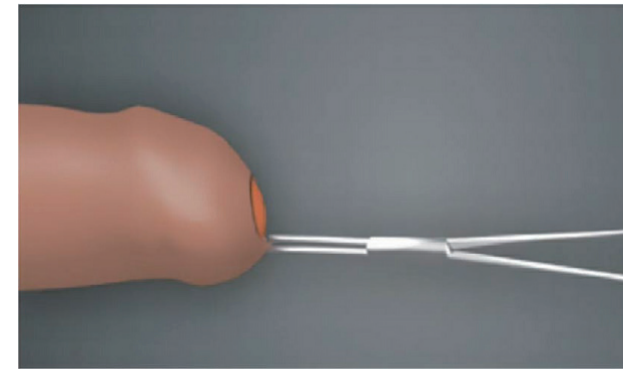


1. Use the hemostatic forceps clamp at
the foreskin frenum place

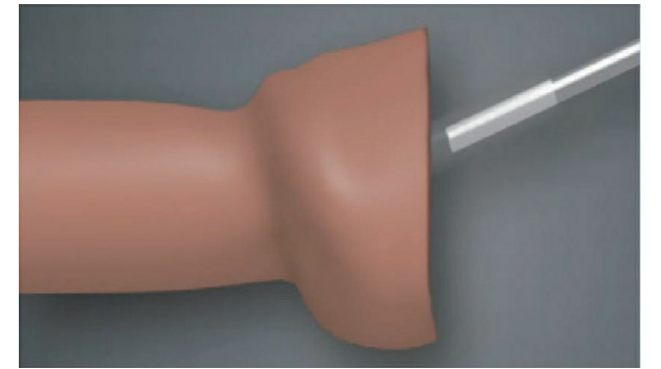


2. Lift up the foreskin, insert the anvil
inside the foreskin and close to glans

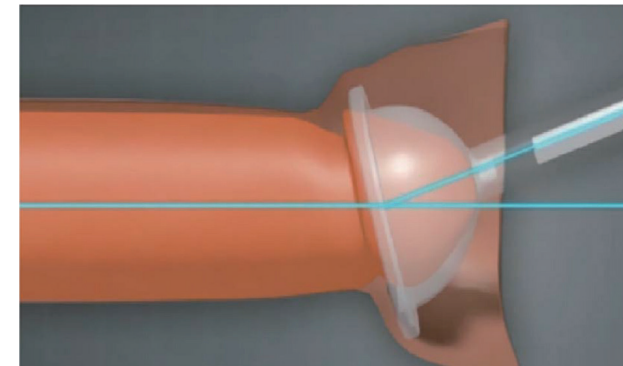


3. Put the anvil longitudinal axis parallel
to the coronary sulcus

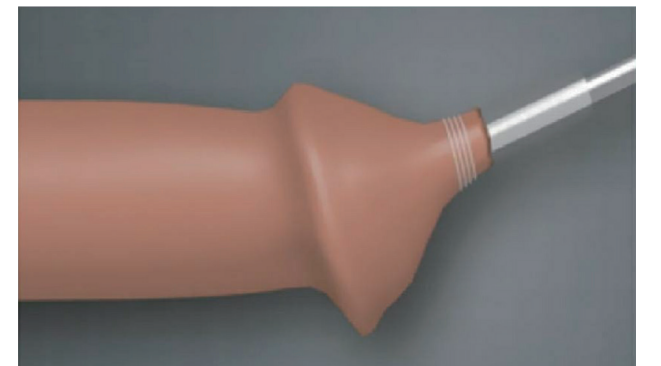

4. Fasten the foreskin to the shaft
with a string

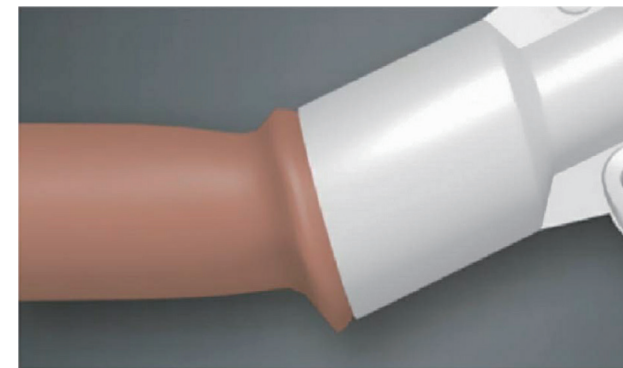

5. Connect the anvil to the instrument,
rotate and tighten the adjustment
knob to the end of the shaft

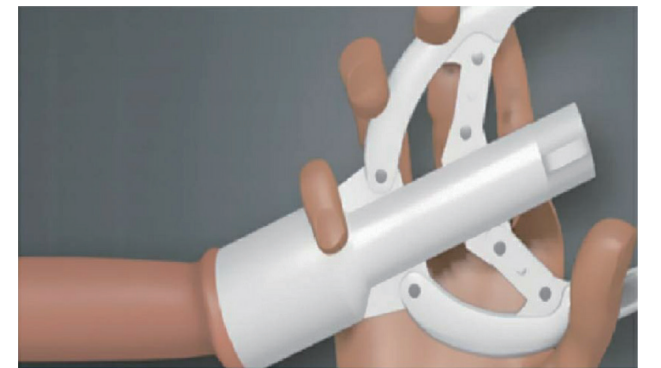

6. Remove the safety, fire the two handles
and keep 5 seconds. Release the instrument,
take out the instrument,
and complete the operation.



Key Features and Benefits

- **Shorten the operation time effectively**
Easy operation, the foreskin is removed and stitched after instrument is fired, the entire operation time is 3-5 minutes
- **Unique anastomosis devices**
Provide much more accurate selection of discretion, convenient operator for accurate positioning the length of the foreskin and organ size
- **Faster healing time**
7-10 days after surgery can heal, postoperative organ appearance neat, smart and natural
- **Provide better postoperative results**
Effectively improve sexual function, prevent male and gynecological diseases, and avoid cross-infection

Ordering Information

Code	O.D(mm) ±1.0	I.D(mm) ±1.0	Staple(QTY)	Height ±0.2(mm)
DCCS-11	19.8	11	9	2.8
DCCS-12	22	12.5	10	2.8
DCCS-15	24	15	11	2.8
DCCS-18	25	17	12	2.8
DCCS-20	27	18	13	2.8
DCCS-22	31	22	16	2.8
DCCS-24	33.5	24.5	18	2.8
DCCS-26	34	25.5	18	2.8
DCCS-28	37.5	28.5	20	2.8
DCCS-30	38	29.5	20	2.8
DCCS-32	41	32	22	2.8
DCCS-34	43	34	22	2.8
DCCS-36	45	36	24	2.8

Disposable Circumcision Stapler



- Sinolinks Medical Innovation, Inc.
- Add.: No.10 Ziwei Road, Zhonglou Zone, Changzhou, Jiangsu, China, 213023
- Tel:86-519-88021690
- E-mail:sales@sinolinks.com.cn
- Web site:www.sinolinks.com.cn